


Teladoc Health™ Mini™ Cart Quick Reference Guide

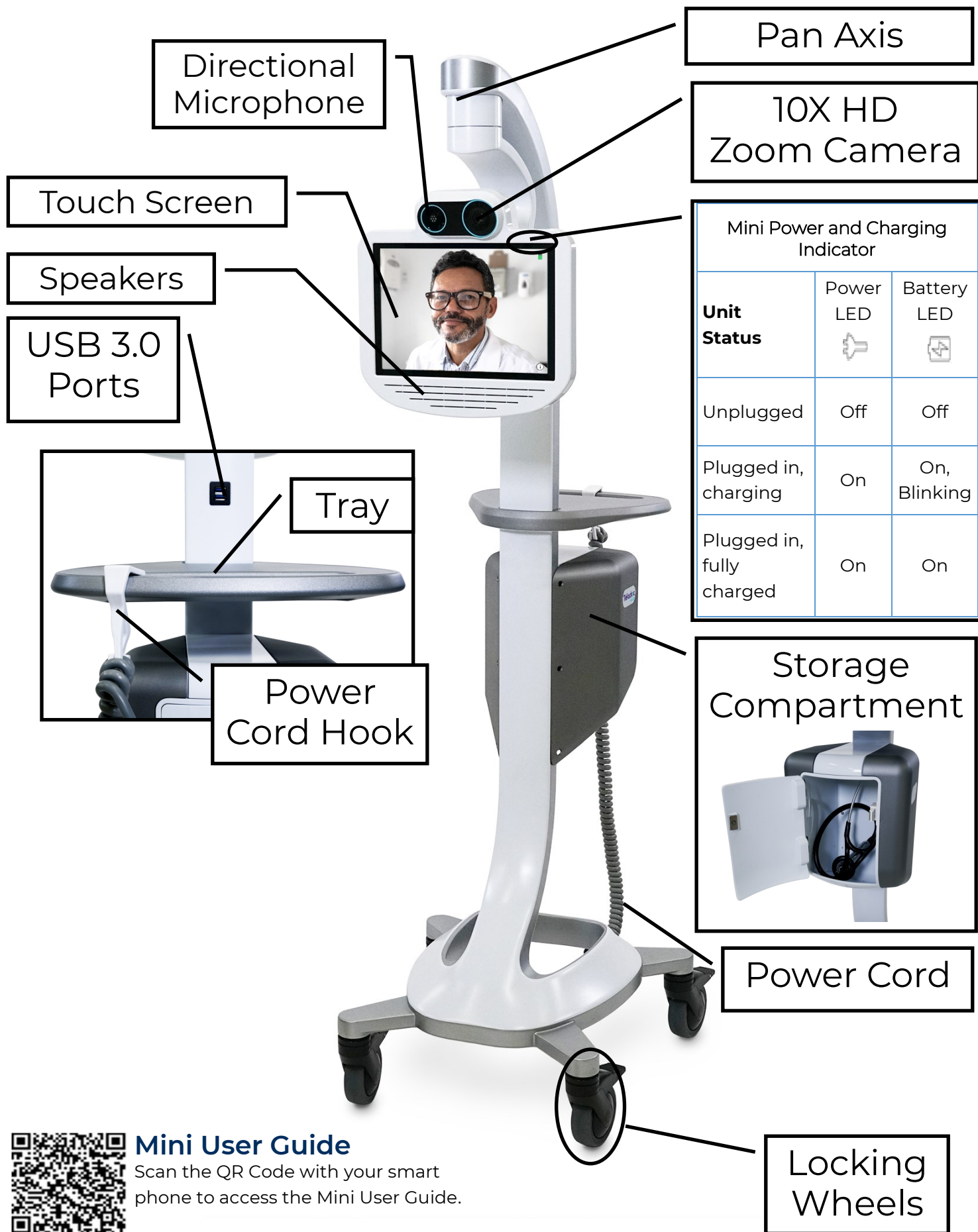

Mini User Guide

Scan the QR Code with your smart phone to access the Mini User Guide.

Teladoc Health™ Mini™ Quick Reference Guide

Powering On the Mini

Press and hold the power button for 2-3 seconds until the device powers on.


Powering Off the Mini

1. Press and hold the power button for 2-3 seconds.
2. Slide the power-off slider on the screen when prompted.

NOTE: It is recommended that the Mini remain powered on to receive updates and to be available for use.

Mini On Screen Navigation


Icon	Function
	Tap to preview any of the cameras connected to the device.
	Tap to open device settings.
	Tap to open the device's User Guide.
	Tap to display information about the Mini

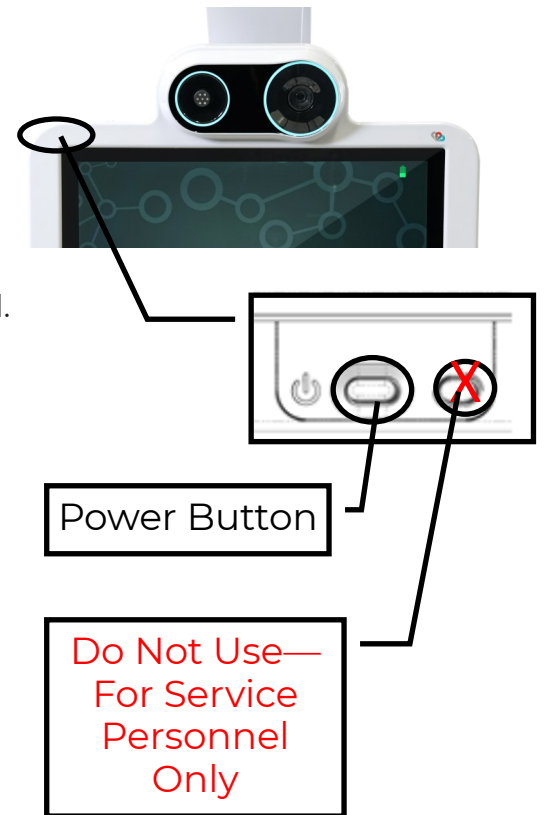

Mini Cleaning Mode

Tap Settings  > Cleaning Mode,  to disable the screen for 30-seconds to allow cleaning.

See the Mini User Guide for approved cleaners.

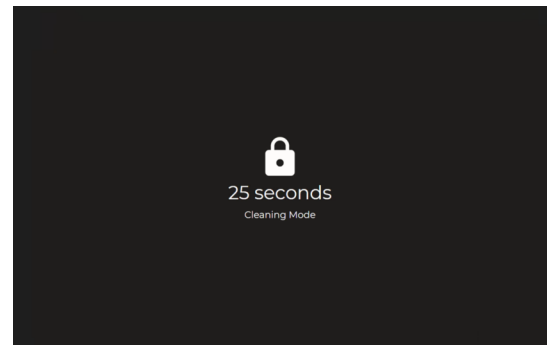

Device Network Checkup

The Network Checkup screen runs and displays the results of a test of the device's current network connection.

Tap Settings  > More Options  > Network Checkup  to run the Network Checkup.

