

Teladoc Health™ Mini™ Wall Mounted

Quick Reference Guide



Mini User Guide

Scan the QR Code with your smart phone to access the Mini User Guide.

Pan Axis

Directional Microphone

Touch Sensitive Display

19x HD Zoom Camera

Mini Power and Charging Indicator

Unit Status	Power LED	Battery LED
Unplugged	Off	Off
Plugged in, charging	On	On, Blinking
Plugged in, fully charged	On	On

Speakers

USB 3.0 Port

Ethernet Port

AC Power Port

Teladoc Health Mini™ Wall Mounted

Quick Reference Guide

Powering On the Mini

Press and hold the power button for 2-3 seconds until the device powers on.

Powering Off the Mini

1. Press and hold the power button for 2-3 seconds.
2. Slide the power-off slider on the screen when prompted.

NOTE: It is recommended that the Mini remain powered on to receive updates and to be available for use.

Mini On Screen Navigation

Icon	Function
	Tap to preview any of the cameras connected to the device.
	Tap to open device settings.
	Tap to open the device's User Guide.
	Tap to display information about the Mini

Mini Cleaning Mode


Tap Settings  > Cleaning Mode,  to disable the screen for 30-seconds to allow cleaning.

NOTE: See the Mini User Guide for approved cleaners.

Device Network Checkup

The Network Checkup screen runs and displays the results of a test of the device's current network connection.

Tap Settings  > Network Checkup  to run the Network Checkup.

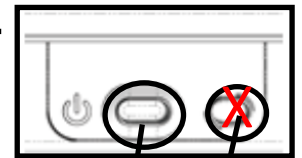

Power Button

Do Not Use—
For Service
Personnel
Only

